






Release Notes

SenseHub Version 6.3 is Released (Dairy & Beef)

New System Features

- **Battery Alerts** for Monitoring Ear Tag Flex V2 is released.
- The **Report Generator** for SenseHub is released. Available only in the Premium application.
- The **SenseHub Discover** is released. Available only for SenseHub Dairy.
- Fields now appear colored/highlighted in reports appearing in the Dashboard Widget just as they do in reports when they are not displayed in the widget.

Animals in Heat 6 					
Health 5 					
608	Group Low		Lactation 1	DIM/Age 353	Health Index 83.6
615	Group Low		Lactation 1	DIM/Age 326	Health Index 85.7
268	Group Calving Preparation		Lactation 3	DIM/Age 341	Health Index 73.1
783	Group Pre-Breeding Heifers		Lactation 0	DIM/Age 645	Health Index 76.3

Important Fixes implemented in this Version

- In previous versions when creating a Sorting Task that repeats for consecutive days, the icons for this task do not display correctly in the sorting scheduler. This is corrected in this version.
- In previous versions when inviting new users the emails are not always translated to the language of the user. This is corrected in this version.
- When moving between graph resolutions, recent event icons may disappear. This is corrected in this version.
- In the previous version event icons might disappear on the mobile app when zooming in or out on graphs. This is corrected in this version.
- In previous versions it was not possible to create a sorting task from a report in the mobile applications. This is corrected in this version.
- In previous versions on mobile devices when entering a new animal, after scanning the flex tag the application direct to the animal card instead of displaying the tag number. This is corrected in this version.
- In the previous version on the mobile app, after changing the language in the mobile app the menus do not immediately change to be displayed in that language. This is corrected in this version.
- In previous versions it was possible to enter an Abortion Event in SenseHub Beef systems. This event is not supported in SenseHub Beef. This is corrected in this version.

Current Supported Herd Management Integrations

These Herd Management Systems are integrated with SenseHub and transfer information using the cloud.

Herd Management System	SenseHub Dairy Version	SenseHub Beef Version
CRV	3.2.0	3.2.0
UNIFORMAgri	2.1.0	3.4.0
DataHub360 – FCEL - IED	6.1.0	6.1.0
Anaborapi – Web Gap	5.2.0	5.2.0
E-exploraco	Not yet supported	4.3.0
Mooml	5.2.0	6.1.0
Vaxa	4.3.0	6.3.0
Farmwizard	6.1.0	
Bovcliv	6.1.0	
ICownect	4.3.0	
ICBF	5.2.0	
Kingswood Computing	5.2.0	
Bovinet	4.3.0	
Ideagri	4.4.1	
Animalia	4.4.1	
Si@lleva	4.2.0	
DCB	2.3.4	
Herde	2.1.0	
SuperKuh	2.1.0	
Faba	3.1.0	
VIT	3.2.0	
Bovisoft	3.2.0	
AWE	3.3.0	
CV France	3.3.0	

These Herd Management Systems are integrated with SenseHub and Transfer information using the Status File Protocol.

Herd Management System	SenseHub Dairy Version	SenseHub Beef Version
Farmsoft	5.3.0	Not yet supported
DC 305	2.1.0	
FEV Plus	3.4.0	
PCDart	2.1.0	
DairyLive	2.1.0	
Agsource DM	2.1.0	
Bovisync	2.1.0	
DHI+	2.1.0	
Dairy Quest	2.1.0	
EasyDairy	2.1.0	
Identity (Jantec)	2.1.0	